HAND-LOG

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DESCRIPTION

The malleable support for hand surgery is made up of an aluminum alloy covered with silicone medical grade (USP class6)

CONDITIONS OF USE

The HAND-LOG device must be used by practitioners surgeons who have received the necessary information about this device .

INDICATIONS

Device for holding the hand in different positions, to be used in the hand surgery. The choice of the use of the HAND-LOG device is the responsibility of the surgeon depending on the type of surgery and the patient's anatomy.

PERFORMANCES

The HAND-LOG device must allow to maintain the position chosen by the surgeon during the surgery. May be hampered by poore use of the device.

PRECAUTIONS

The HAND-LOG device is supplied non-sterile and must be cleaned, decontaminated and sterilized in autoclave steam temperature 134 ° C (± 3 ° C), time 5 minutes, pressure 2 bar. After use the HAND-LOG device must be washed and decontaminated. Prior to sterilization, the HAND-LOG device must be manually restored in its original form, avoiding any maneuvering that could damage the internal aluminum foil breakage or silicone coatings that may cause the foil to appear. Chlorine, acid and abrasive cleaning and decontamination solutions are not recommended.

WARNING !

The HAND-LOG device should not be used under the following conditions: internal aluminum foil breakage, rupture of the silicone coating.

EXPOSURE TO THE ENVIRONMENT

Store in a dry place away from heat sources.

DISPOSAL

The device must be disposed of as Hospital waste.

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