# LARK'S

SURGICAL DEVICES

 dispositivo medico fabbricato in Italia  
medical device manufactured in Italy

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## **ENDOLOG**

System for correction  
of hallux valgus

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*Special thanks to Prof. Roberto Aldegheri and Prof. Carlo Biz of Padua University - Orthopedic Clinic, for their collaboration.*

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Endolog system is proposed for the mini-invasive surgical treatment of hallux valgus deformity correction of the follow entities:

- Lieve / Slight: IMA < 11°; MTF < 20°
- Moderato / Moderate: IMA > 11° < 16°; MTF > 20° < 40°
- Grave / Severe: IMA > 16° < 18°; MTF > 40°

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*The Endolog system is composed of a curved component (intramedullary nail) manufactured in three different sizes and a angular stability screw in two different lengths.*

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*The main feature of the system is referred to the curvilinear component (nail) which, when introduced into the canal after metatarsal osteotomy, is opposed to three distinct points: on the lateral cortex with regard to the apex of the curvature near the proximal metaphysis with regard to the distal part of the plant and surface lateral of metatarsal head for the flat plate-shaped plant, ensuring a excellent stability in absence of rotation or tilting in the metatarsal canal.*

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The angular stability screw, introduced in the metatarsal head through the hole of the plate of the nail is the final element locking in stability of the implant.

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*The angular stability screw Ø 3,66 mm length 15 and 20 mm.*

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*The use of Endolog implant in the correction of hallux valgus, thanks to the characteristics listed above, the elastic stabilization of the head of the first metatarsal, important corrective displacement are possible with good development of periosteal-type callus in absence of correction loss, which is a problem observed with other techniques of stabilized distal osteotomy, allowing early mobilization metatarsal phalangeal joint.*

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The implant provides a summary stable but not rigid distal osteotomy with:

- significant improvements in all clinical parameters assessed in the pre-operative;

The implant also ensures a rapid normalization post-operatively with:

- 1) no pain
- 2) good alignment
- 3) disappearance of the areas of plantar hyperkeratosis
- 4) no joint stiffness

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The pre-operative planning should be assessed on radiographs in the upright position and assessing the following parameters:

- L'angolo inter metatarsale / *Inter Metatarsal Angle (IMA)*
- L'angolo metatarso falangeo / *Metatarsal Phalangeal Angle (MTF)*
- L'angolo articolare superficie prossimale / *Proximal Articular Surface Angle (PASA)*

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*One very important step concerns the exostosectomy, which must be done at first always cheap as going to affect the degree of correction of the inter-metatarsal angle.*

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*Exostosectomy must be made by taking distal-proximal wedge shape with a micro sagittal saw, in order to allow the plant to correct any valgus of the distal phalanx, thereby avoiding a second surgical act.*

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*This surgical procedure allows the correction of the P.A.S.A. (Fig. 1).*

*If during the exostosectomy, it was necessary to spin clockwise or counterclockwise the metatarsal head on the front section, you will have to guide the sagittal saw blade in oblique on the dorsal plantar direction (fig. 2).*


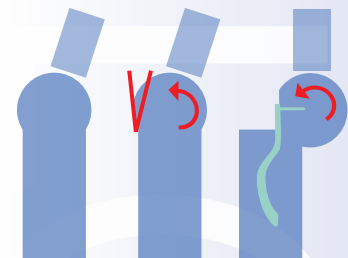
Fig. 1

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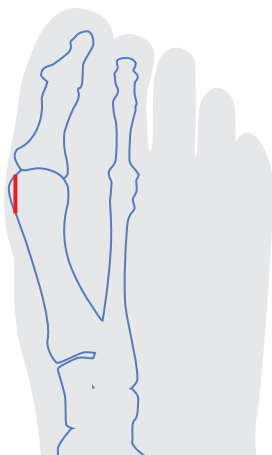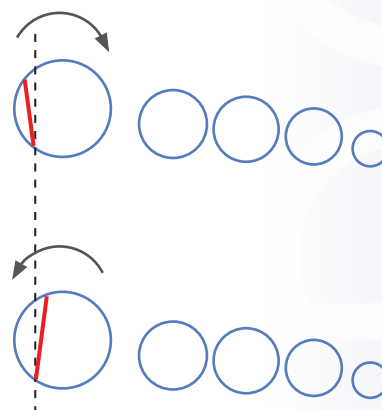
*The patient is under local regional anesthesia with tourniquet at level of foot neck.*

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*Longitudinal performance of 2,0 / 2,5 cm incision at medial level of the first metatarsal.*

*The vascular-nervous bundle is isolated and protected.*

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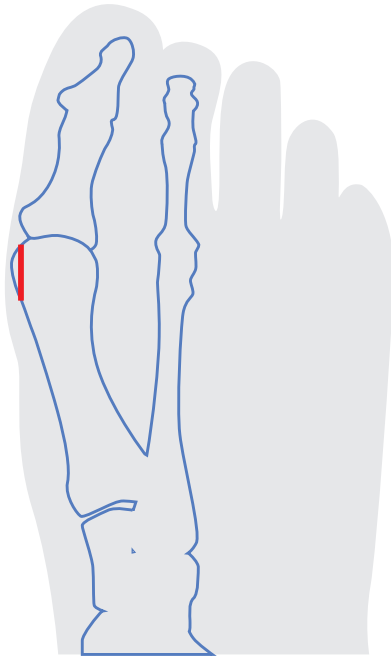
Esostosectomia cuneiforme disto-proximale da 2-4 mm a zero.

*Disto-proximal exostosectomy made by taking a wedge shape from 2-4 mm to zero mm.*

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
*The purpose of this important step essentially allows you to restore the valgus of distal phalanx in addition PASA once the flat surfaces of the metatarsal head is resting on the flat part of the implant. (see description of "PRE-OPERATIVE PLANNING")*

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
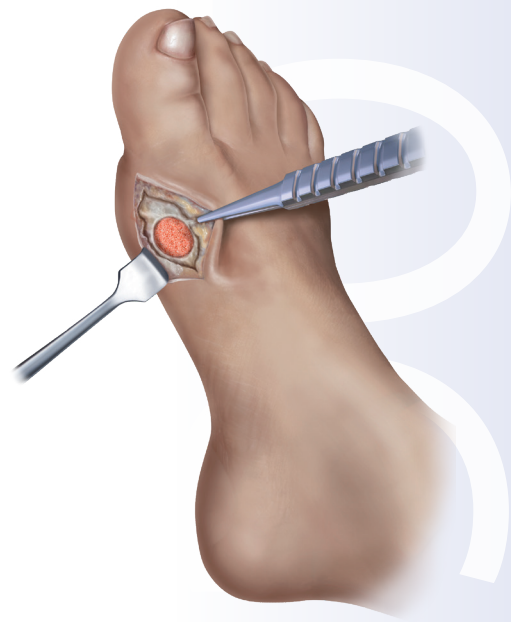
*Linear Mitchell osteotomy (by a micro sagittale saw) performed at the level of the distal collum of the first metatarsal bone according to pre-operative planning.*

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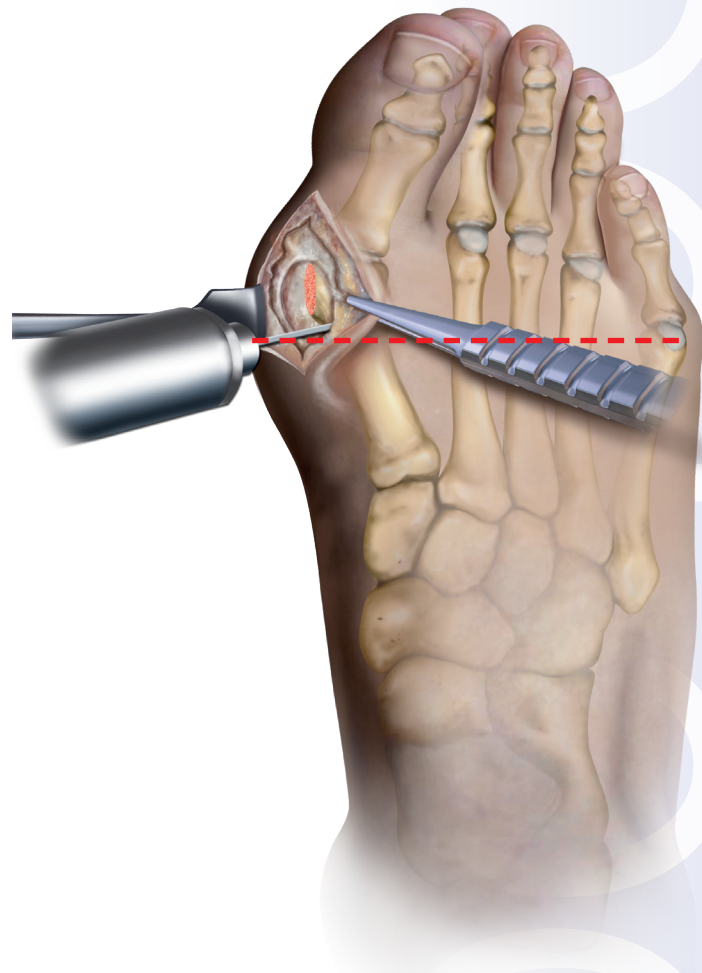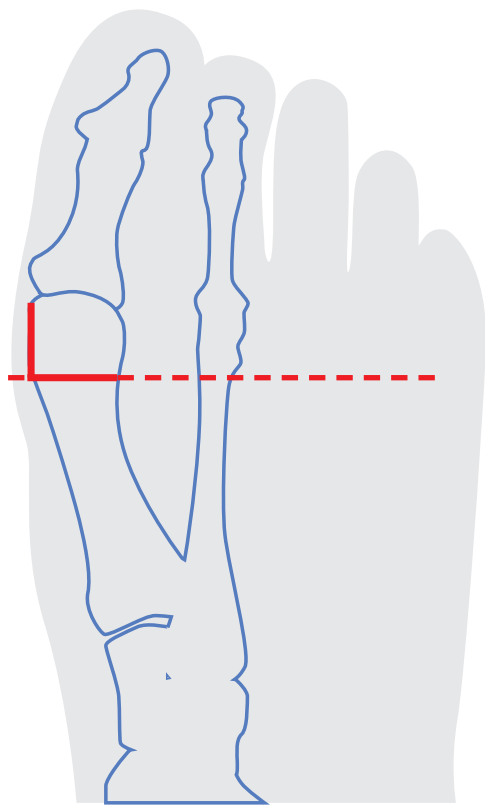*If you need to lengthen or shorten the metatarsus, metatarsal osteotomy will be slightly skewed.*

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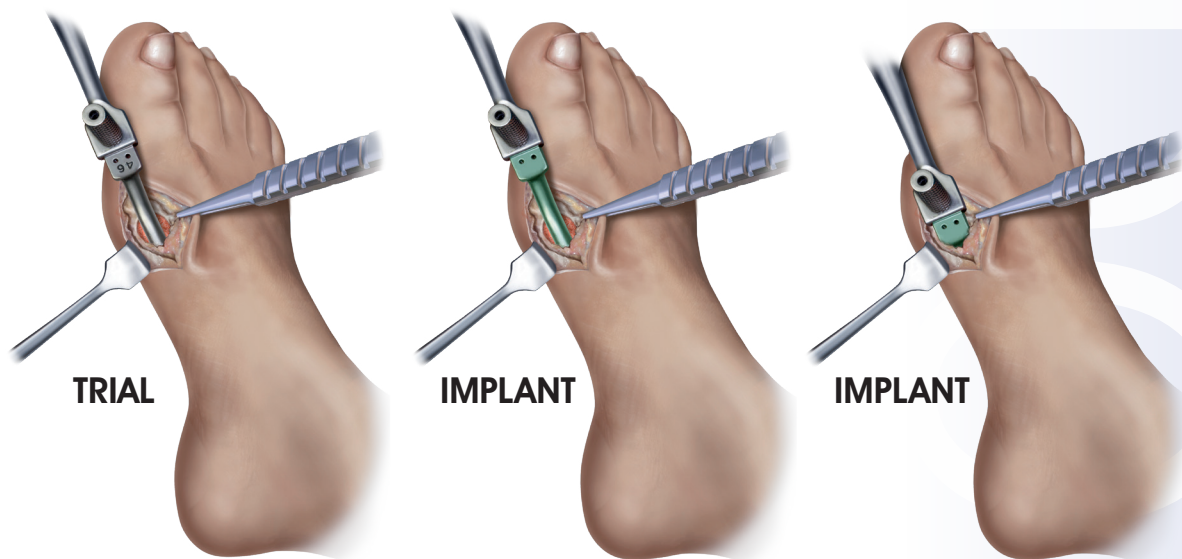
*Implant trial test, starting from size 46, fitting the same on the dedicated impactor by the drilling guide.*

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*If the implant size 46 does not give guarantee for stability in the medullary, use the system size 45.*

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*Where the canal metatarsal is not sufficiently long, shorter than 30 mm to accommodate the trial facility, to use the system size 44.*

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*Once you have determined the optimal size of the plant, final assembly implant, assembling the same on the dedicated impactor by the drilling guide.*

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**The introduction of the system must be as much as possible perpendicular to the longitudinal axis of the metatarsus, preventing rotation of the same during the introduction.**

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In order to facilitate the early introduction of the system will be necessary in the canal metatarsal diastasis longitudinally rhyme manually exercising manual traction osteotomy of the distal phalanx.

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*During the insertion of the implant will occur the automatic translation of the metatarsal head with respect to the metatarsus. To complete the insertion of the implant until the longitudinal axis of the drill guide is aligned with the center of the metatarsal head.*

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*To check that the metatarsal head, during the introductory phase of the implant remained in the correct position, namely that did not have unwanted locations.*

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*Keeping manual corrections listed above, to proceed to the pre-stabilization of the implant inserting one or two 1,0 mm diameter K-wire in the metatarsal head through the micro holes of the implant, located just below the drilling guide.*

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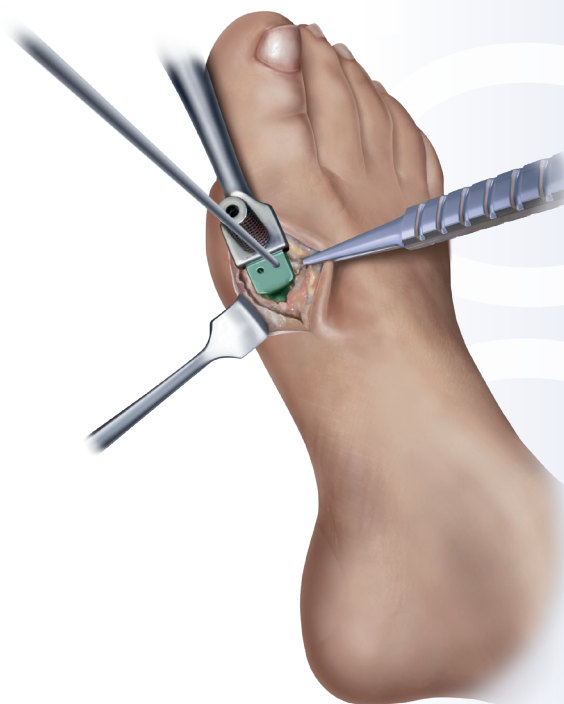
*To make the preparation of the stabilization screw house, with graduated tip mounted on orthopaedic drill and inserted through the hole of the cylindrical guide.*

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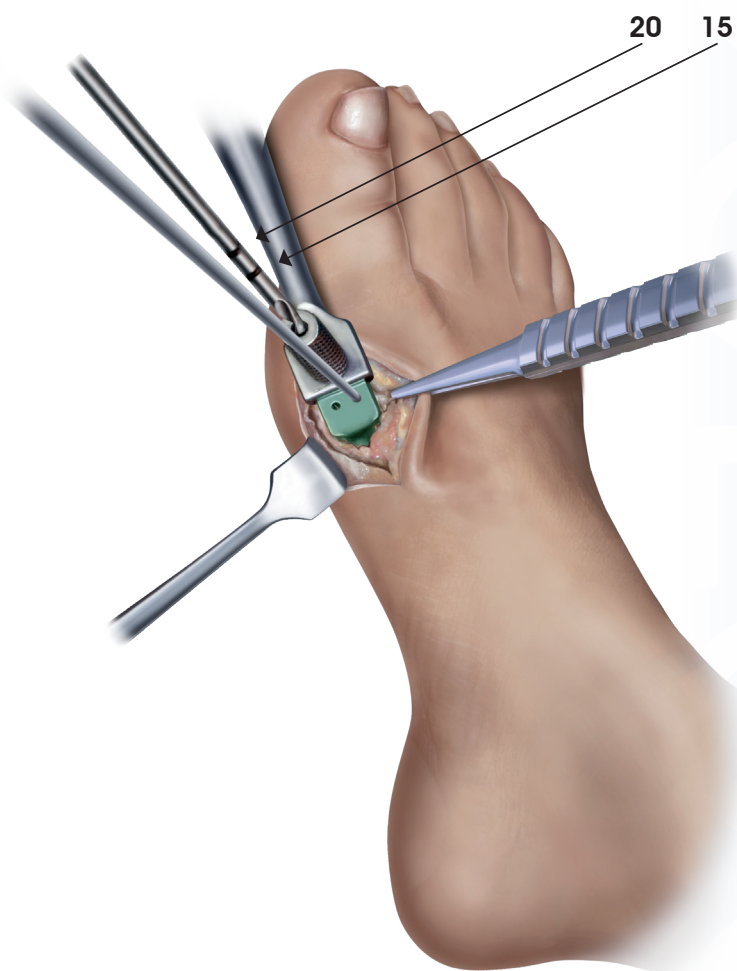
*The two notches of the graduate tip to be utilized: 15 or 20 mm.*

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*The drilling will be up to and no beyond than the cortical side of the metatarsal head.*

*The lenght of the screw to be used will be that one on the graduate tip notch, which has reached the proximal point of the drilling guide.*

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
*During this step it is critical to keep the parallax of the screw with respect to the surface of the implant plate in order to allow proper grip micro threading under the head of the screw thread to the implant plate.*

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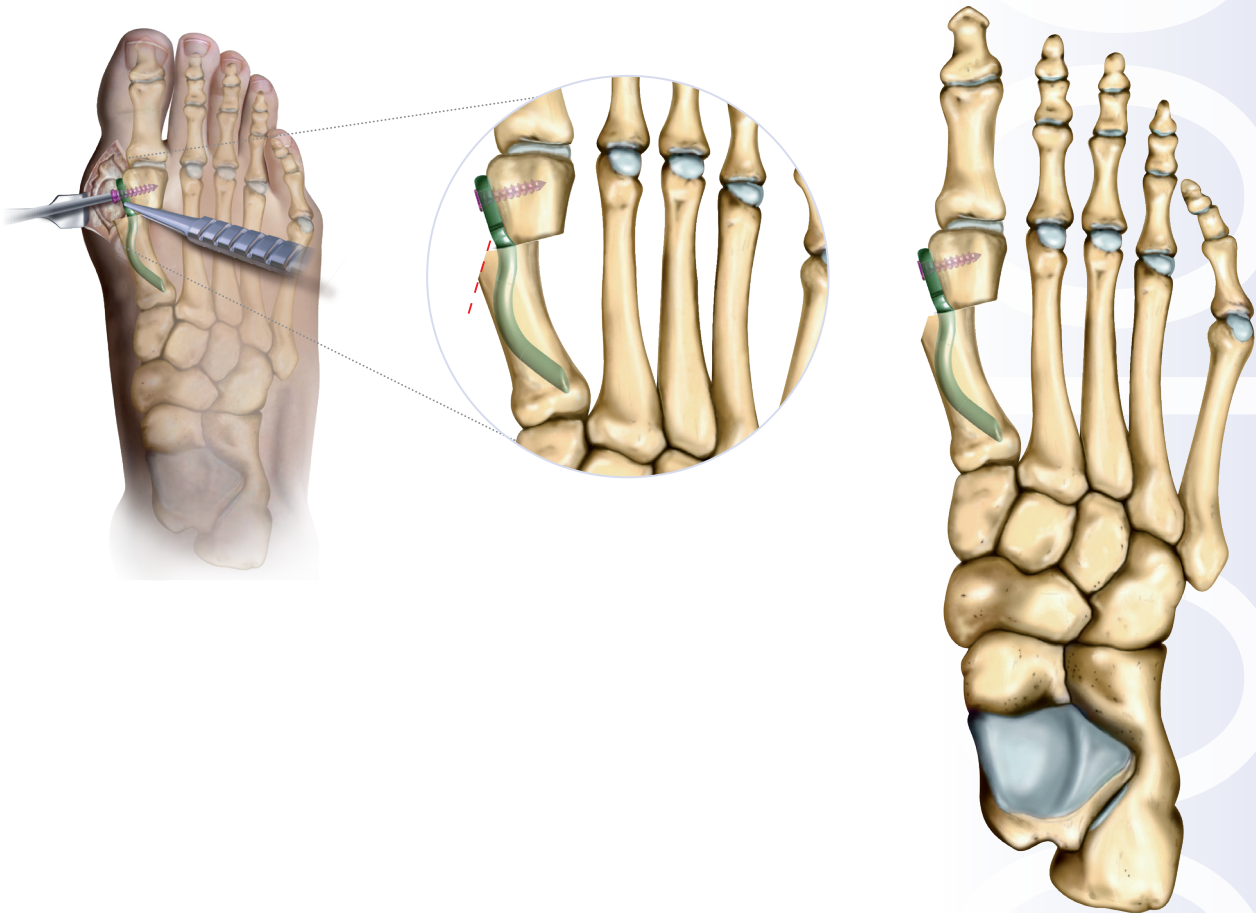*Before suturing the capsule, to smooth the angle of the metatarsus in order to avoid frictions with the soft tissues.*

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*Suture of the capsule (could be useful if the case so requires, to put again tension on the capsule).  
Removal of the tourniquet, hemostasis control, and completion of the suture.*

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## **POST-OPERATORIO / POST-OPERATIVE**

### **CARICO IMMEDIATO / IMMEDIATE LOAD**

Con calzatura larga confortevole  
*With large and comfortable footwear*

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*Starting with delicate flexion-extension movements of the joint, further enhancement of rehabilitation.*

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Removal of the implant will be effected using the same route with the following surgical procedures:

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*Titanium alloy Ti6Al4V ELI - ASTM F 136 - ISO 5838 - 5832-3  
Surface treatment by anodic oxidation.*

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Sterility is guaranteed only if the package is undamaged.  
Do not re-sterilize. SINGLE USE ONLY.

## BIBLIOGRAFIA / BIBLIOGRAPHY

### **J Orthopaed Traumatol (2009) 10 (Suppl 1):S41-S60**

Hallux valgus mini-invasive surgery: comparison of two techniques - R.Valentini, G. Fancellu  
*Orthopaedic and Traumatologic Clinic, Trieste University (Trieste, IT)*

### **8th EFAS Congress - 2nd-4th September 2010 - CMU, Geneva, Switzerland**

A comparison between Endolog technique and Percutaneous Distal Metatarsal Osteotomy (PDO)  
Carlo Biz, Roberto Aldegheri  
*Padua University - Orthopedic Clinic (Padua, IT)*

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### **J Orthop Surg Res - 2015 Jul 2;10:102. doi: 10.1186/s13018-015-0245-1.**

Endolog technique for correction of hallux valgus: a prospective study of  
30 patients with 4-year follow-up  
Carlo Biz, Marco Corradin, Ilaria Petretta, Roberto Aldegheri  
*Padua University - Orthopedic Clinic (Padua, IT)*

### **Int Orthop - 2019 Mar;43(3):625-637. doi: 10.1007/s00264-018-4138-x. Epub 2018 Sep 14.**

Minimally invasive surgery for hallux valgus: a systematic review of current surgical techniques  
Francesc Malagelada, Cyrus Sahirad, Miki Dalmau-Pastor, Jordi Vega, Rej Bhumbra,  
Maria Cristina Manzanares-Céspedes  
*Royal London Hospital, UK*

### **Foot Ankle Surg - 2021 Jan;27(1):46-51. doi: 10.1016/j.fas.2020.02.001. Epub 2020 Feb 5.**

The Endolog technique for moderate to severe hallux valgus treatment: Clinical and radiographic  
analysis of 194 patients  
Federico Bertolo, Andrea Pautasso, Claudio Cuocolo, Daniela Invernizzi, Francesco Atzori  
*Turin University - Orthopedic Clinic (Turin, IT)*

### **Foot Ankle Int . 2021 Apr;42(4):409-424. doi: 10.1177/1071100720969676. Epub 2020 Dec 15.**

Functional and Radiographic Outcomes of Minimally Invasive Intramedullary Nail Device (MIIND) for Moderate  
to Severe Hallux Valgus  
Carlo Biz, Alberto Crimi, Ilaria Fantoni, Jacopo Tagliapietra, Pietro Ruggieri  
*Padua University - Orthopedic Clinic (Padua, IT)*

MedicalDue s.r.l.

**LARK'S**  
SURGICAL DEVICES

#### **Produttore e distributore / Manufacturer and distributor:**

Medical Due S.r.l.  
Via Trento, 43  
IT-37014 Castelnuovo del Garda (VR) - Italy  
Tel. +39 (0) 45 6450777 - fax +39 (0) 45 8525203  
www.medicaldue.it  
P.IVA IT-02895150239 - Reg.Imp. VR1998-63861  
R.E.A. 292702 - Cap. soc. Euro 10.400,00 i.v.